

# Installation Guides

## ME series

(Mobile equipment carts)



ME010




MEN011




ME111




# A. How to assemble the aluminum pole and base ?

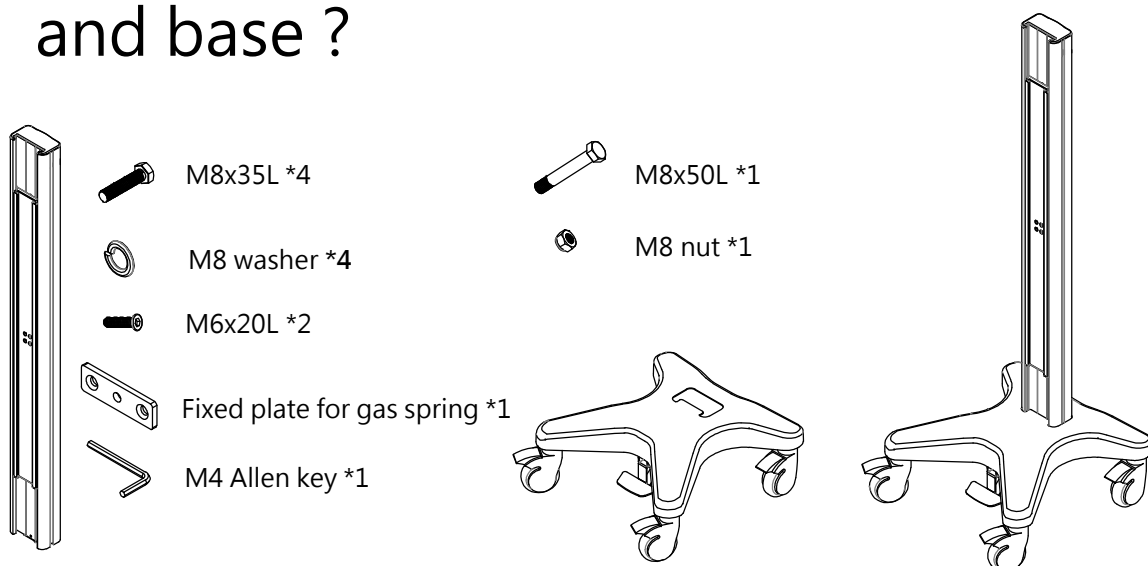

STEP. 1 Please hold the inner pole to avoid sliding down.

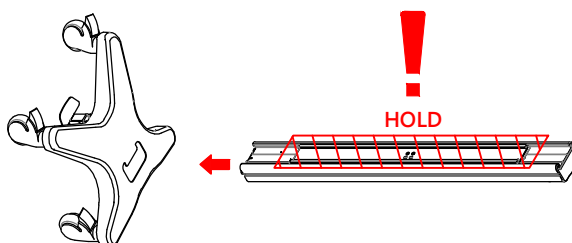


STEP. 2 Push the step up, screw M8 and the plate of gas spring by M6. Push back the step and screw M8 nut.

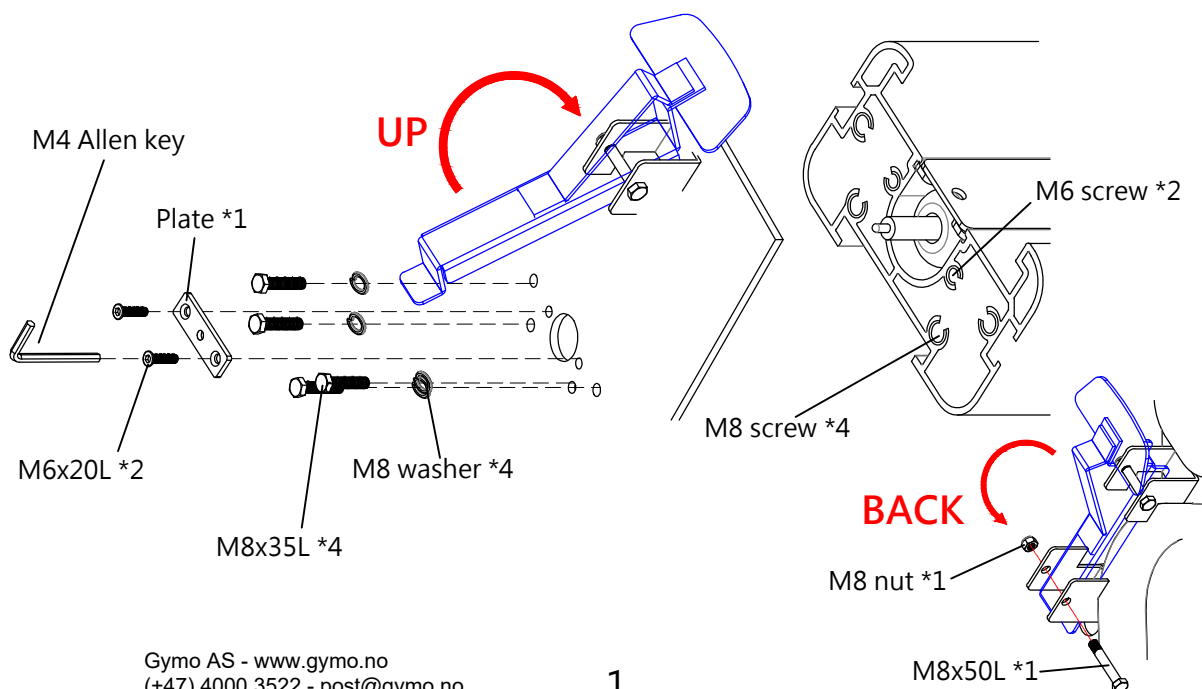


## B. How to assemble the shelf ?

### STEP. 1 Fix the slide block M6x10L.

Please do not screw it tightly for adjusting the height on the aluminum pole.

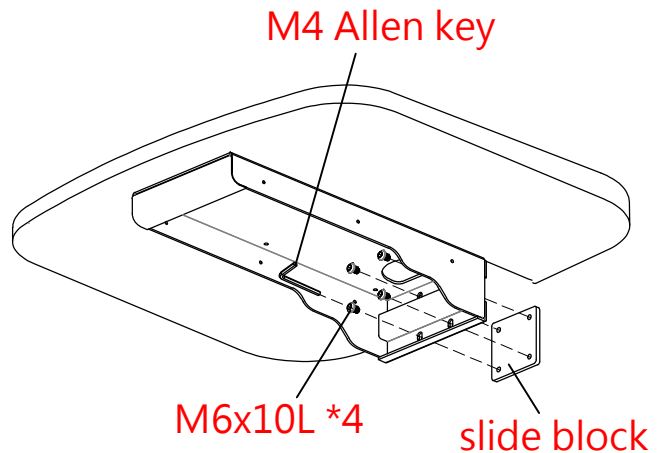


### STEP. 2 Slide the shelf in the aluminum pole at proper position and screw it tightly.

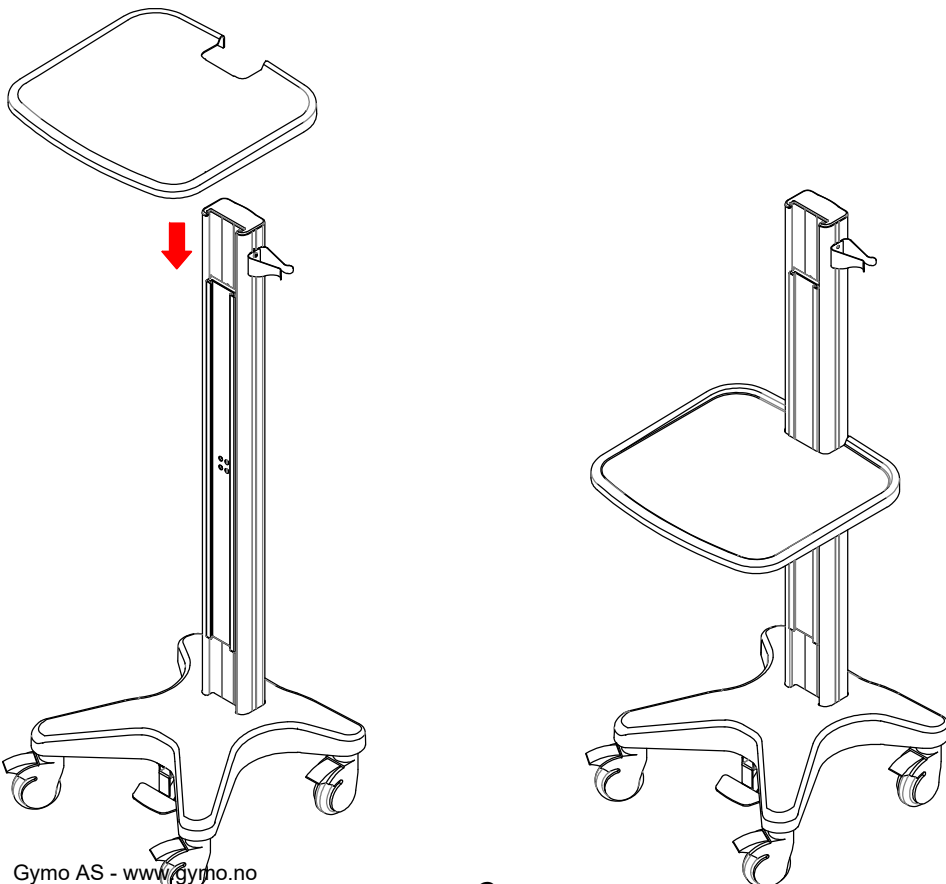


# C. How to assemble the scanner holder ?

STEP. 1 Please assemble scanner holder L & R together and leave a space for adjustment.

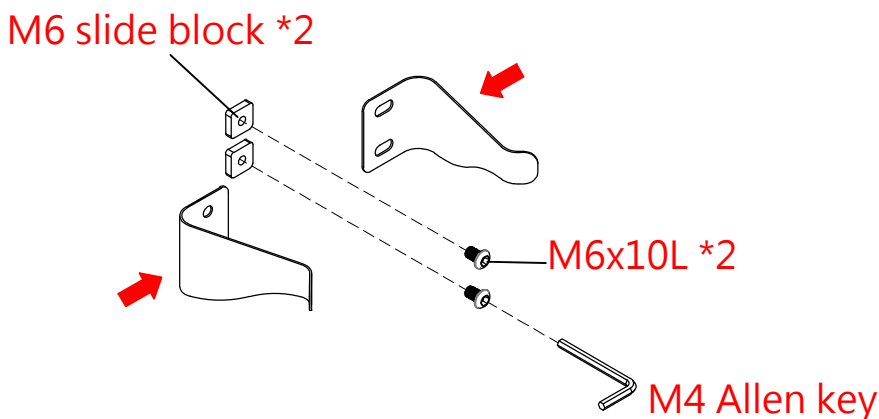

STEP. 2 Please remove the pole top cover and slide the scanner holder in the pole to assemble it at proper position.

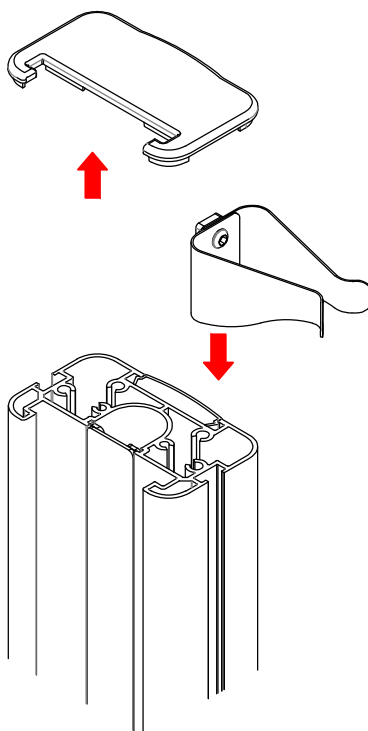


STEP. 1 Please remove the pole top and back cover.

STEP. 1 Please remove the pole top and back cover.




STEP. 3 Please change the back cover from length 405 to 365mm.

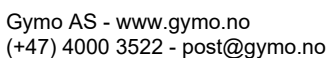


# E. How to assemble the keyboard holder & mouse home ?




STEP. 1 Please turn the shelf upside down to assemble rails.

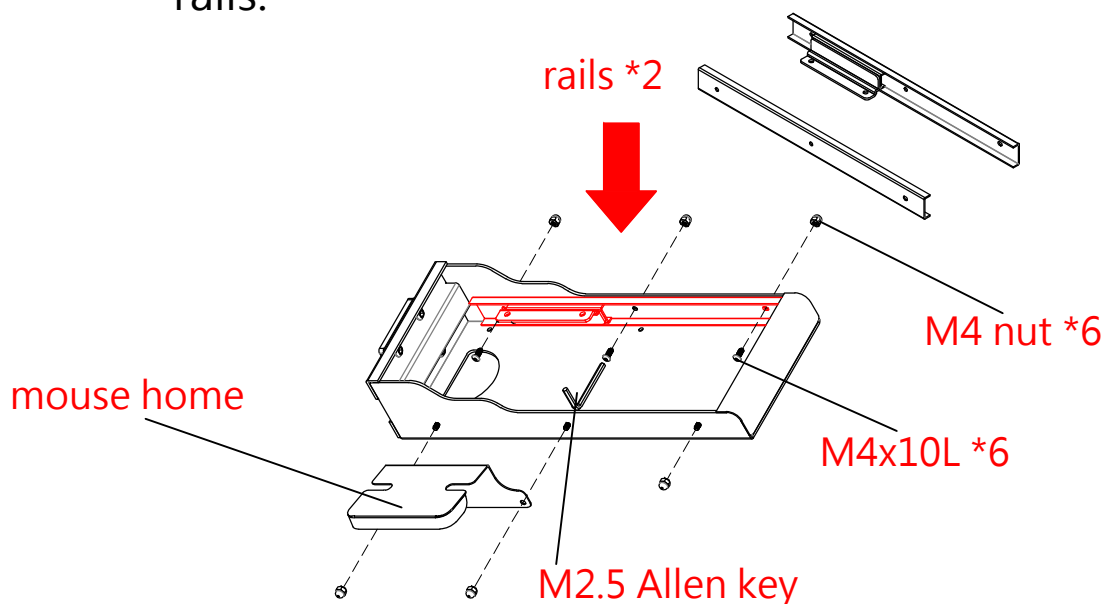


STEP. 2 Please assemble the keyboard holder on rails.

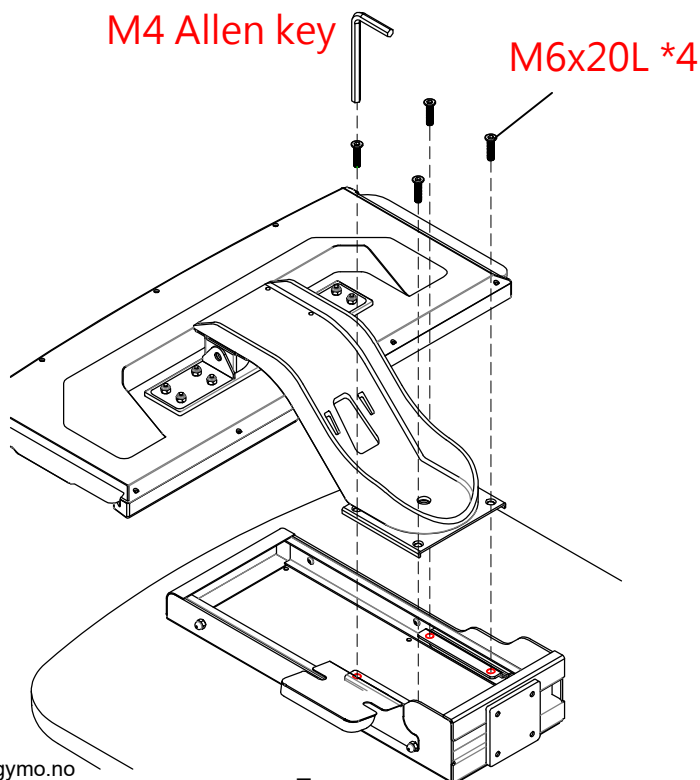


## F. How to assemble the screen holder ?

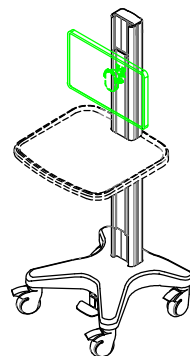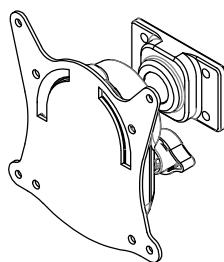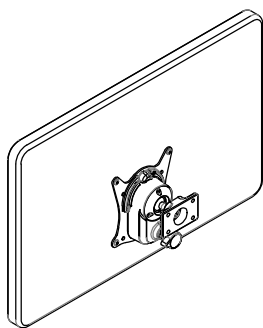

STEP. 1 Please Slide the screen holder in the pole and screw it at the proper position.


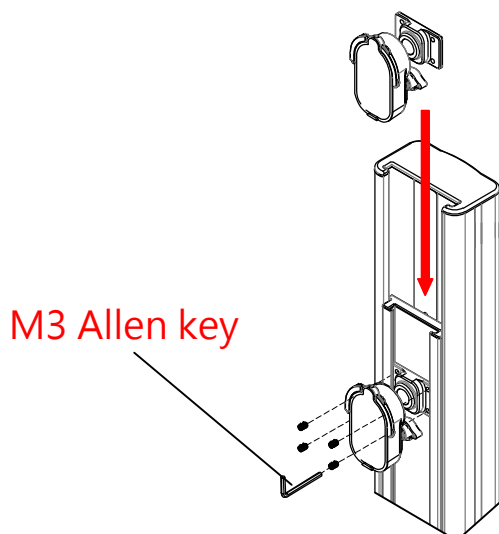


M6x10L \*4

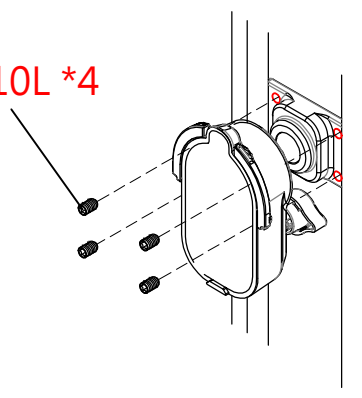

STEP. 2 Please assemble the fixed plate at the screen back.

STEP. 3 Please push the bottom and turn 90 degrees to fix well.

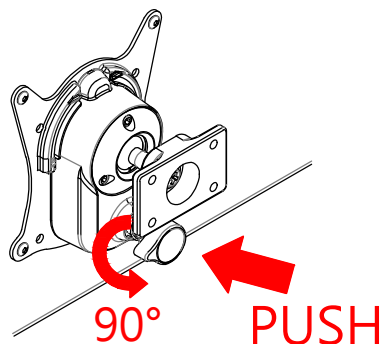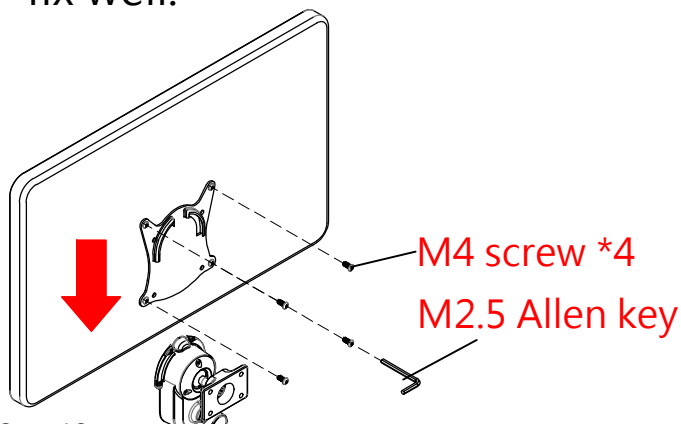



[www.BAILIDA-MEDICAL.COM](http://www.BAILIDA-MEDICAL.COM)



Machan International Co.,Ltd.  
No.65,Lane 706,Erkan Rd.,waipu Dist.,  
Taichung,Taiwan

