

Step by step guide for the grinding of medication with the Severo Electric Medicine Grinder

Follow this recommended grinding protocol to avoid unwanted exposure to medication dust
Suivez ce protocole de broyage recommandé pour éviter une exposition indésirable à de la poudre médicamenteuse
Siga este protocolo de molido para evitar una exposición no deseada a polvo de medicamento
Befolgen Sie dieses empfohlene Zerkleinerungsprotokoll, um sich nicht ungewollt Medikamentenstaub auszusetzen.

To avoid the risk on exposure and cross contamination please follow the recommended protocol on the other side of this leaflet. Alternatively check the following tutorial online at www.severo.nl/demovideo.



1

Open the Severo by pushing the at the bottom of the grey button at the front. Put the medication in the cup, max till first line. Place the Severo ronudel centered on top of the cup



2

Place the grinding head on top of the roundel und push it downward in the cup. Put the cup together with the grindinghead in the Severo.



3

Close the cap, if the machine is switched on, the grinding will start automatically. Otherwise push the grey button to start the grinding process. At the end of the cycle the button will light up green.



4

Open the cap and carefully take out the grindinghead from the cup. Hold the roundel with one finger in the cup. Check for visual contamination on the grinding head.



5

Lift the roundel and discard the roundel. If necessary wipe the roundel over the edge of the cup.



6

By using the roundels, the grindinghead is ready to use again immediately. In case of tubefeeding grind twice! Now the medication can be mixed with our Severo swallowing gel or with food.

Requirements

• **SeveroCup** (article code: MED001)

The following additional items are recommended for use during medication grinding with the Severo:

• **SeveroRoundel** (article code: MED003)

For keeping the grinding head clean. The use of roundels prevents cross contamination and saves cleaning time after each grinding cycle.

• **SeveroCup Lid** (article code: MED002)

For completely sealing off the cup, this way minimizing spills and contamination risks during grinding and medication distribution.

Cleaning

- With visable contamination, thoroughly clean the grinding head with an alcohol cloth.
- If visably contaminated the entire Severo can be cleaned with a damp cloth and some detergent.

Tip

Always check the contents of the SeveroCup if the SeveroRoundel was damaged during grinding. The SeveroRoundel has been extensively tested with the hardest kinds of medication. Despite of this it being damaged during grinding can never fully be ruled out.



Medisinsk utstyr og helseartikler